

*Twist-a-Dose*<sup>®</sup>

## A different “twist” on transdermal applications.

Twist-a-Dose<sup>®</sup> may be the perfect alternative when you have a hard time getting your pet to take oral medications. The transdermal applicator gives you easy, simple and accurate dosing every time.

### Twist-a-Dose is:

➤ **EASY.**

No more struggling to read those tiny dose lines!

➤ **SIMPLE.**

Allows you to apply directly to your pet’s ear!

➤ **ACCURATE.**

Twisting gives you the correct dosage... every time!

Instructional video available at:  
**[WedgewoodPetRx.com/  
ww/education-fun](http://WedgewoodPetRx.com/ww/education-fun)**



See reverse side for instructions.



wedgewood  
P H A R M A C Y

GREAT  
CARE,  
DELIVERED.™



To avoid contact with the medication, wear protective gloves when administering.

### **The first time you use the Twist-a-Dose transdermal gel applicator:**

1. Remove the protective cap covering the white applicator tip.
2. Turn the end until a small amount of gel forms evenly at the tip to prime the pen. This may take several rotations.
3. Turn the end with the left-facing arrow until it lines up with the right-facing arrow.
4. Wipe the applicator tip clean with a tissue.
5. Replace the cap if you are not administering medication. Otherwise, continue with step 2 under "Regular Dosing."

*This process should only be done the first time the applicator is used. Subsequent uses should begin with step 1 under "Regular Dosing." If medication does not appear during a subsequent use, follow the instructions above.*

---

### **Regular Dosing:**

1. Remove the protective cap covering the white applicator tip.
2. Turn the end with the left-facing arrow a full two revolutions until it realigns with the right-facing arrow (for a 0.05ml dose), or as prescribed by a veterinarian.
3. Place the applicator tip directly on the inside portion of the ear flap (inner pinna) and spread it over the entire surface using the applicator tip. Be sure to gently rub the medication in well using the applicator tip.
4. Wipe the applicator tip clean with a tissue.
5. Replace the cap.
6. Alternate ears with each dose.
7. Gently clean the residue from the ear with warm water on a cotton ball or soft tissue, just prior to the next application of medicine.
8. Wash hands after cleaning ears.

**Don't hesitate to contact us at 800.331.8272  
if you have any questions.**